

Established 1978

**Physician's
Mobile
X-Ray, Inc.**

Physician's Mobile X-Ray, Inc.

945 East Park Drive

Suite 102

Harrisburg, PA 17111


Phone: (717) 561-4940 Fax: (717) 561-4467

Instructions for Burning an Ultrasound CD

Open a web browser and navigate to:




<https://app.corestudycast.com/coreweb/users/login>

Enter username and password and click "Login"

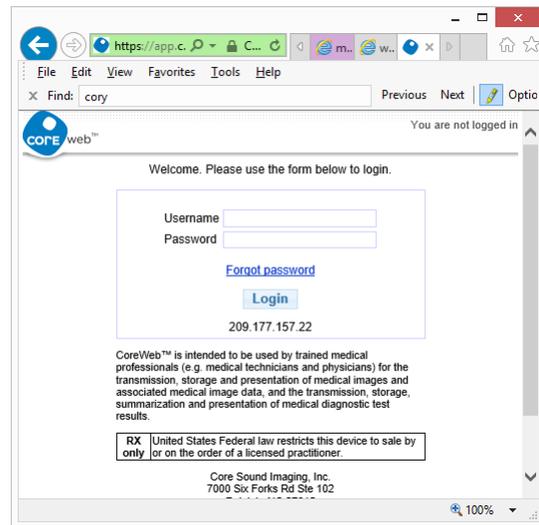

Click on "Select criteria", and select "Patient Name".

Enter name of patient to search. Click on "Search"

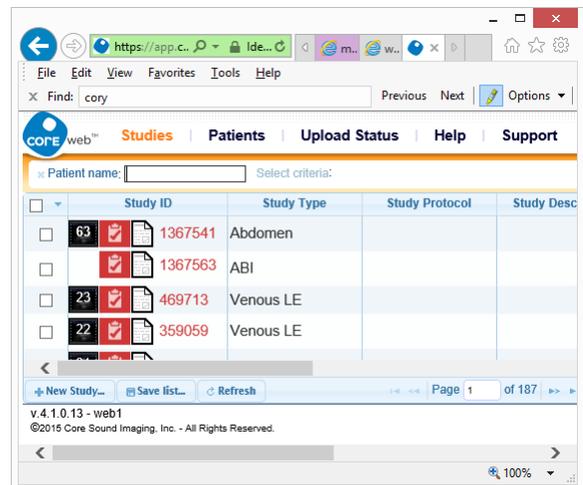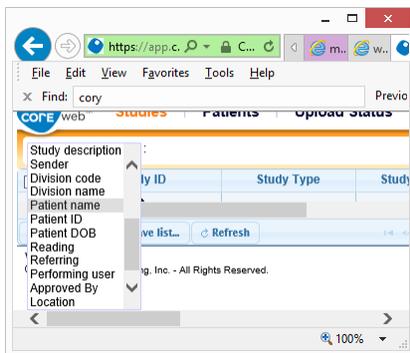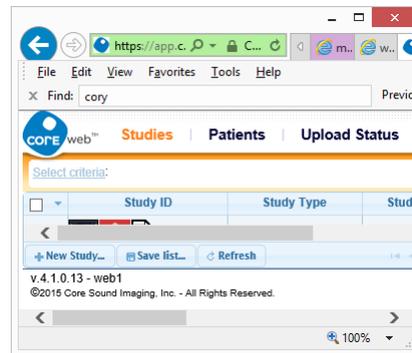



Established 1978

**Physician's
Mobile
X-Ray, Inc.**

Physician's Mobile X-Ray, Inc.


945 East Park Drive

Suite 102


Harrisburg, PA 17111


Phone: (717) 561-4940 Fax: (717) 561-4467

Click on the checkbox next to the study/studies you wish to burn to CD.

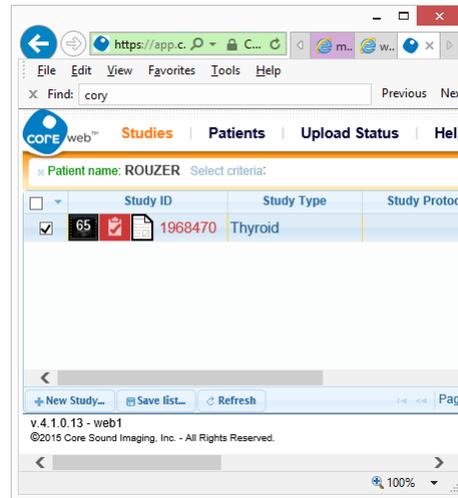

Click on the dropdown arrow at the top of the list and select "Archive"

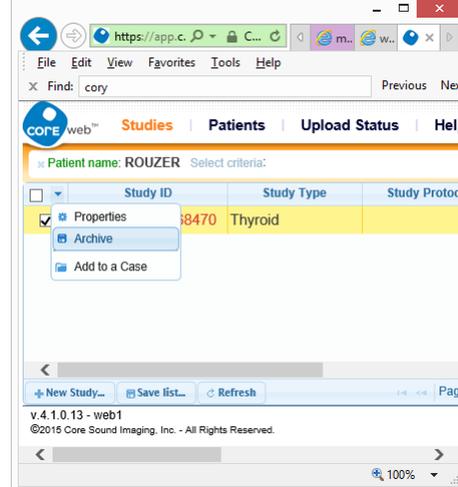

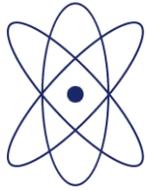

Established 1978

**Physician's
Mobile
X-Ray, Inc.**

Physician's Mobile X-Ray, Inc.


945 East Park Drive

Suite 102

Harrisburg, PA 17111


Phone: (717) 561-4940 Fax: (717) 561-4467

Use the default setting in the dialog box and click on "Archive as ISO Image with CorePlayer".

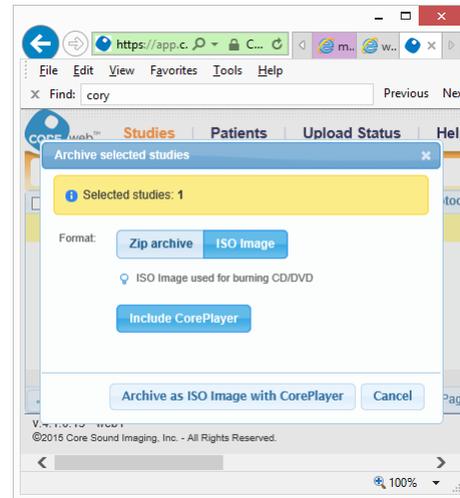

When prompted, click on "Save As" and save the file to the Desktop.


To burn the CD, you will need to download FreeISOBurner. If you do not already have the program on your Desktop, you can download it from the PACS by clicking on the Help link at the top of the page.

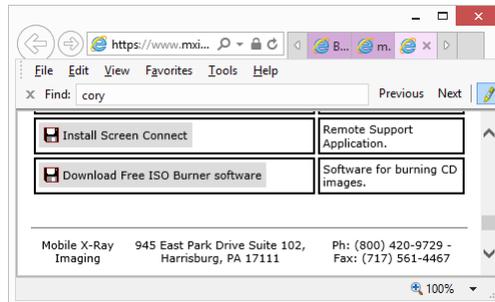


Insert a blank CD into the computer.

Run FreeISOBurner.exe

Click "Open". Browse to the .iso file you downloaded.

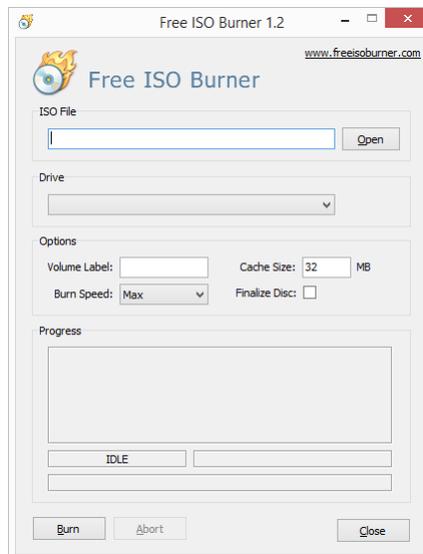

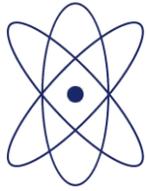

Established 1978
**Physician's
Mobile
X-Ray, Inc.**

Physician's Mobile X-Ray, Inc.
945 East Park Drive
Suite 102
Harrisburg, PA 17111
Phone: (717) 561-4940 Fax: (717) 561-4467

Click "Open". Browse to the .iso file you downloaded.

Click on Burn.

