

## Deleting Studies/Images (Viztek)

To free disk space, you must delete studies from the Viztek. This is a two-step process: mark the study for deletion, and purge the deleted studies.

► **To mark a study for deletion**

1. Right-Click on the study in your Study List.
2. Left-Click on **Delete Study** from the pop-up menu.

► **To purge your deleted studies**

1. On the **Study** menu, select **Purge**.
2. In the **Purge Study** window that appears select each study you wish to purge.
3. Left-Click on the **Purge Study** button.
4. To confirm your choice, click **Yes**.